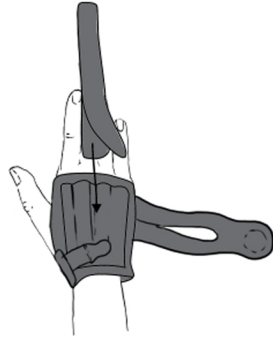
KOD KATALOGOWY	AM-D-03
NAZWA HANDLOWA	ORTEZA PALCA REKI
MODEL	ORTEZA PALCA REKI AM-D-03
CODE	AM-D-03
TRADE NAME	FINGER SPLINT
MODEL	FINGER SPLINT AM-D-03

POLSKI

ENGLISH

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ADDITIONAL FINGER IMMOBILIZER

ATTENTION: BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTION COMPLETELY AND CAREFULLY AND CHECK IF PRODUCT'S ELEMENTS ARE SEEDY. ONLY UNDAMAGED DEVICE AND ITS CORRECT USING ARE VITAL TO THE PROPER FUNCTIONING OF IT. SAVE THE PRODUCT INFORMATION, TO BE ABLE TO REREAD THE INSTRUCTION LATER.

PRODUCT DESCRIPTION:

Reh4Mat products are innovative and are carefully made of the high quality materials. Our key objective is customer's satisfaction. In case of any questions feel free to contact with us by e-mail: biuro@reh4mat.com. **IMPORTANT:** - In case of skin lesions occurring and allergic reaction is it necessary to contact with physician.

Is available as an additional equipment to the following spine braces:

-AM-D-03

IMPORTANT:

- Proper use of the splint requires diagnosis by physician or physical therapist.
- Optimal splint functioning can only be achieved by selecting the right size. The Reh4Mat splint must be fitted by an authorized vender, physician or educated psycho-therapist.
- It is necessary to control using the splint, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in place, where the splint touches should be clean (the creams or ointments should not be used).
- Is it necessary to control the skin's condition and in case of skin lesions occurring is it necessary to contact with physician.
- In case of skin lesions (epidermis scrapes, scatches or eczema) using of the splint is allowed after dressing applying.
- Chronic using the splint may cause limitation of the active and passive movement in joints. If there are no contraindications, it is recommended to exercise several times during the day. Exercises are selected individually for each patient by a master's degree or technician rehabilitation as physician has recommended.
- In the case of poor blood circulation in the limbs, a splint may cause hyperaemia. Avoid applying the splint too tightly in the beginning.
- Use of the splint during the night should be done only if this was prescribed by your doctor.

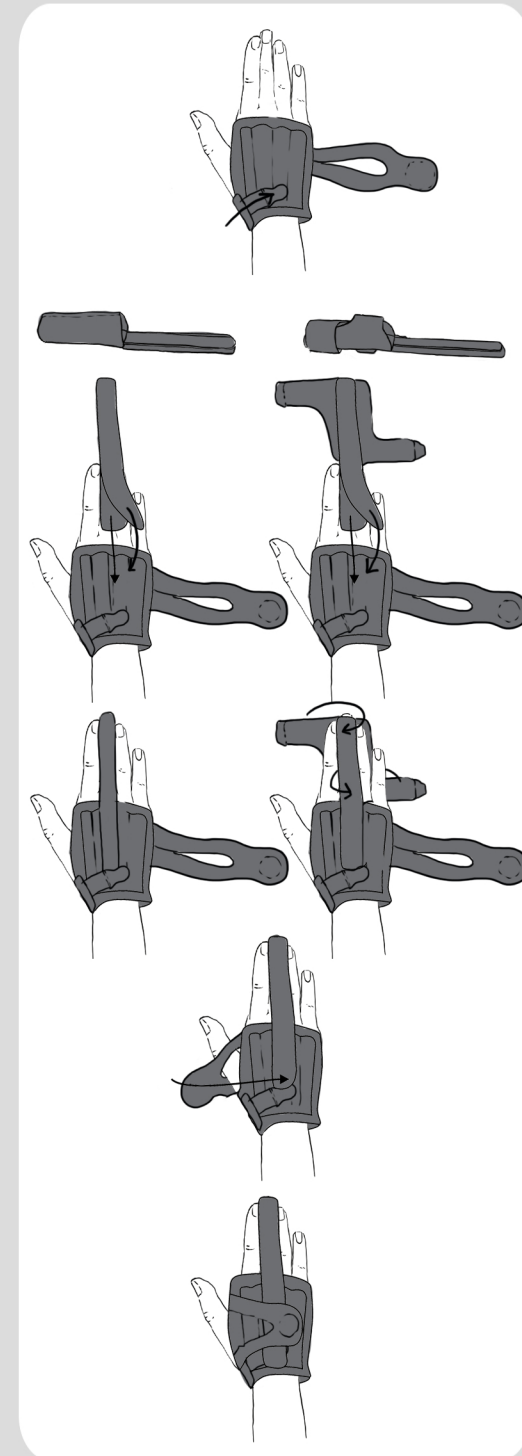
CARE AND MAINTENANCE:

- Wash by hands in warm water with soap in temperature of 30 C,
- Sluice carefully,
- Do not iron,
- Do not chlorinate,
- Do not dry clean,
- Do not tumble dry,
- Protect from moisture,
- Protect from high temperatures,
- Dry away from sources of warmth.

CAUTION: IF THE DEVICE HOLDS LATERAL SPLINTS OR SHELLS AND IT IS POSSIBLE TO DISMANTLE THEM WITHOUT ANY TOOLS, BEFORE WASHING REMOVE THEM.

Te product is available for AM-D-03 finger immobilizer only.

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ATTENTION: Pain is always the sign of warning. If the pain maintains or increases it is advisable to consult your doctor or physical therapist.
ATTENTION: SPLINT SHOULD BE USED ONLY BY ONE PATIENT. USING OF THE DEVICE NOT ACCORDING TO THE MANUAL MAY CAUSE SERIOUS COMPLICATIONS.