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## ENGLISH

The brace is an excellent solution for effective therapy in case of injury or other conditions. Reh4Mat products are appreciated and recommended by specialists (doctors, physiotherapists) and patients. Our products are produced in Poland and made of high quality materials. In case of any questions, feel free to contact us: [export@reh4mat.com](mailto:export@reh4mat.com).

## INDICATION

The elbow orthosis is used for stabilization and immobilization, if required, of upper limb joints, including the elbow. The product is useful in case of orthopaedic injury and other neurological and rheumatologic conditions.

## CARE AND MAINTENANCE

- Product should be maintain in cleanliness and use according the manufacturer recommendations.
- Product should be maintain in cleanliness and use according the manufacturer recommendations. Clean the product systematically, at least once a month or more if ten if necessary, in accordance with the Manufacturer's recommendations, ensuring that it is kept in good condition.
- If the product holds lateral splints or shells and it is possible to dismantle them without any tools, remove them before washing.
- Wash by hands in warm water with soap in temperature of 30°C.
- Sluice carefully, use when completely dry.
- Do not iron, do not chlorinate, do not dry dry clean, do not tumble dry.
- Keep it in a dry and clean zone.
- Protect from moisture, extreme temperatures, sunlight, dry away from sources of warmth.

## CONTRINDICATIONS

- The product should be used according to the recommendations of aphysician or physiotherapist
- Do not use the product in the wrong size.
- Only a correctly fitted size, in accordance with the size table, available on the product packaging and the manufacturer's information materials, guarantees effectiveness and safety.
- Do not use the product by people and children with mental disorders without guardian control.
- Do not use the product on patients with superficial and deep sensation disorders without the supervision of a guardian.
- The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occurring, it is necessary to contact a physician.
- In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of orthosis is allowed only after contact with physician..
- Do not use the product against its purpose of use.
- Do not use the product in case of damage or shape's deformation.
- Do not attempt to service this product yourself. Independent interference with the design of the product may result in permanent damage to the user's health.
- Do not use the accessories which are not recommended by manufacturer.
- Do not use the product near an open flame or on a hot surface. This can seriously injure the user and lead to serious damage to health.
- Use the orthosis during night should by done only if this was prescribed by your doctor.

## SAFETY RULES

- Attention! The product should be used according the manual's recommendations. If you have any questions, please consult your care professional.
- Attention! Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety and effectiveness. If you have any questions, please consult your care professional.
- Attention! Using of the product not according to the manual may cause serious complications.
- Attention! Please, keep the manual for future.
- Attention! Check the product's condition. If the product is damaged, discontinue use and contact the manufacturer. Only an undamaged product and proper use guarantee its right and safe working.
- Attention! Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using.
- Attention! Manufacturer is responsible for product's effectiveness and safety only in case of purchasing the product in the authorized seller.
- Attention! Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor or physical therapist.
- Attention! For single patient use only.
- Attention! The product should be used according to the recommendations of a physician or physiotherapist.
- Before first using, check the size of the product. Only good size guarantees safety and effectiveness. A wrong size of the product decreases effect of treatment, increases discomfort and wrong position of the user. Proper use of the brace diagnosis by physician or physiotherapist.
- Control using the product, especially in children, people with memory dysfunction, psychiatric and people with sensory disturbances.
- The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occurring, it is necessary to contact a physician.
- In case of varicose veins, Deep Vein Thrombosis (DVT), oedema the using of product is allowed only after contact with physician..
- Before using remove package and accessories like papers, cards or labels.
- Do not use the product against its purpose of use.
- Do not use the product in the wrong size.
- Do not use the product by people and children with mental disorders without guardian control.
- Do not use the product in case of damage or shape's deformation.
- Do not attempt to service this product yourself.
- Do not use the accessories which are not recommended by manufacturer.

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\*\*\*

Serious incidents involving this device should be reported both to manufacturer and to the competent authority in your country.

Please follow the setting up included with the product.

A list of all available spare parts is available at [www.reh4mat.com](http://www.reh4mat.com), for the selected product.

NOTE - Instruction manual in other languages are available on the website [www.reh4mat.com](http://www.reh4mat.com)

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